

# COMMON TEST STRIP PROBLEMS

- Be aware that some things in drugs may cause a false positive.
- If you are drug checking with an FTIR, use the instrument if you think there is a false positive

## TYPE OF TEST STRIP

## POSSIBLE CROSS-REACTING SUBSTANCE

Medetomidine

Levamisole

Nitazene

Caffeine

Xylazine

Lidocaine

Benzodiazepene

None known. Make sure sample dissolves in solution. OK to shake/stir well!

Fentanyl

Methamphetamine, MDMA, Diphenhydramine (Benadryl)